**Corporate Head Office
& Manufacturer:**

Ambu A/S
Baltorpbakken 13
P.O. Box 232
DK-2750 Ballerup
Denmark
Tel.: +45 72 25 20 00
Fax: +45 72 25 20 50
E-mail:
ambu@ambu.com
www.ambu.com

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



RESPIRATORY CARE

Ambu† and Ambu® are registered trademarks of Ambu A/S, Denmark.
Ambu A/S is certified according to ISO 9001 and ISO 13485.

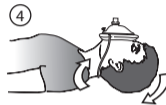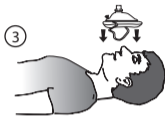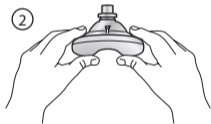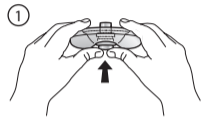English

Directions for use.....4

Deutsch

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Français

Mode d'emploi.....8

Italiano

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The Ambu Res-Cue Mask™ provides a physical barrier between the rescuer and victim, eliminating direct contact of the rescuer's lips with unknown subject.

The Res-Cue Mask promotes an airtight seal to the face allowing ventilation through both the mouth and nose simultaneously. Proper training in ventilation and CPR should be obtained before using this device. Practice using your Ambu Res-Cue Mask on Ambu manikins so that you are ready for any emergency that might occur.

NOTE: In the event that you do not have your Ambu Res-Cue Mask; START mouth-to-mouth ventilation without delay.

USE ON ADULTS AND CHILDREN:

1. Remove the mask/valve assembly from case. Push out the soft dome with your fingers and ensure that the one way valve is in place.
2. Put on the protective gloves if available.
3. Clear the airway of any debris.
4. Place the mask over the patient's mouth and nose. Use the rim of the mask to retract the lower lip so that the patient's mouth remains open under the mask. Use thumbs to hold the mask and your fingers to grasp the lower jaw in front of the earlobes. Pull upward to open the airway and seal the mask.
5. Blow slowly into the airway valve twice and watch as the patient's chest rises. If the patient's chest does not rise; reposition the head and chin until you successfully ventilate the patient.
6. Ventilate the patient every five seconds for an adult or every three seconds for a child. Continue rescue breathing until the patient responds or you are relieved by a higher medical authority.

NOTE: If the patient vomits; remove the Res-Cue Mask and clean both the patient's airway and the mask with your gloved fingers. If you should contaminate either the valve mouthpiece or any of your exposed skin surfaces, use the alcohol wipes to clean these areas immediately. Resume rescue breathing promptly.

USE ON INFANTS:

The Ambu Res-Cue Mask may be reversed to better fit an infants face. Place the nose section of the mask under the infants chin. Seal the Res-Cue Mask as described under the Adult/Child section and ventilate at least every three seconds using the previously described technique.

USE WITH OXYGEN (OPTIONAL):

The Ambu Res-Cue Mask may be used with supplemental oxygen. Attach the oxygen tubing to the optional oxygen inlet and set the flow rate. A head strap is included on all models that have the optional oxygen valve. If the patient is spontaneously breathing, use the head strap to position the Res-Cue Mask on the patient's face.

If the patient is NOT breathing, begin mouth to mask ventilation immediately. The use of supplemental oxygen to the Ambu Res-Cue Mask will increase the delivered oxygen to the patient. The rescuer will not be affected by the use of supplemental oxygen.

USE WITH OTHER VENTILATION EQUIPMENT:

The Ambu Res-Cue Mask may be used with the Bag-Valve Mask ventilation devices. Simply detach the valve/filter assembly and attach the Res-Cue Mask to the device.

CLEANING:

The Ambu Res-Cue Mask is reusable and easy to clean. Wash the mask in warm soapy water. Rinse in clean water. Use a 1:64 bleach to clean water mixture to soak the mask for ten minutes. Rinse the mask again with fresh water and allow to air dry. The face mask can also be disinfected using an activated glutaraldehyde solution (such as Cidex). The one-way valve and filter housing is intended for single patient use and should be discarded after use.

DO NOT pasteurize, boil, steam or gas autoclave this device.

TECHNICAL SPECIFICATIONS:

Ambu Res-Cue Mask is in conformity with Council Directive 93/42/EEC concerning Medical Devices

Temperature Limits: Operating: -40°C (-40°F) to 70°C (158°F)

Storage: -18°C (-1°F) to 50°C (122°F)

Inspiratory Resistance: 2.5 cm/H₂O at 50 lpm flowrate

Expiratory Resistance: 3.5 cm/H₂O at 50 lpm flowrate

Mask in Carrying Case: Dimensions: 13,5 cm x 11 cm x 4,5 cm Weight:

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